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The emergence profiles and screw channels, as applicable, must be prepared by suitable means (e.g. wax, silicone covers).

Cementation of root posts (Visalys[®] CemCore and Visalys[®] Core) and indirect core build-ups (Visalys[®] CemCore)

- Prepare the root canal, root post or indirect core build-up according to the instructions for use from the relevant manufacturer.
- Wet the cleaned and dried surfaces of the root canal, the coronal dentin and the enamel parts, as applicable, completely with Visalys[®] Tooth Primer and rub in for 20 sec. Remove excess from the root canal with a paper tip and dry all surfaces with a gentle jet of air.
- Apply Visalys[®] CemCore or Visalys[®] Core according to the instructions for use of the product and cement the root post or indirect core build-up (only Visalys[®] Core) with the respective adhesive. Do not light cure Visalys[®] CemCore.
- Perform the build-up of the core as described under "Instructions for use for core build-up" from the Visalys[®] CemCore or Visalys[®] Core instructions for use.

Application Instructions as Dental Hard Substance Primer for Core Build-Up

- Clean and dry the cavity.
- Ensure close to the pulp must be protected with the appropriate means.
- Apply a clean and dry operating field (rubber dam) during the entire application.
- Selective phosphoric acid etching of the enamel may be undertaken optionally.
- Drop Visalys[®] Tooth Primer into a preparation tray. Light protection is the primary concern.
- Apply Visalys[®] Tooth Primer to the entire adhesive surface of the core or the cavity using a suitable applicator and rub in for 20 sec. Any excess can be removed with a cotton pellet and blown with air. Then dry with a jet of air. The preparation should be processed within 5 min., as constituents can evaporate.
- Perform the build-up of the core as described under "Instructions for use for core build-up" from the Visalys[®] CemCore instructions for use or the Visalys[®] Core instructions for use.

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Stand der Information: 10. November 2023

ENGLISH
<p>Constituents</p> <p>The main components of the Visalys[®] Tooth Primer are water, acidic monomer (10-MDP) and HEMA.</p> <p>Trademarks</p> <p>Visalys[®] Tooth Primer, Visalys[®] Core Try In Paste, Visalys[®] Core and Visalys[®] Core are registered trademarks of Kettenbach GmbH & Co. KG.</p> <p>Note</p> <p>Not all Kettenbach products are available in all countries.</p> <p>For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to use by or on the order of a dentist.</p> <p>If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.</p> <p>Limitation of Liability</p> <p>Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from the use of this material, whether direct, indirect, special, in-tervening or consequential, even if the manufacturer has been advised, including warranty, contract, negligence, or strict liability.</p> <p>The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and instructions in connection with any use of the product.</p> <p>Updated: November 10, 2023</p>

ENGLISH
<p>Constituents</p> <p>The main components of the Visalys[®] Tooth Primer are water, acidic monomer (10-MDP) and HEMA.</p> <p>Trademarks</p> <p>Visalys[®] Tooth Primer, Visalys[®] Core Try In Paste, Visalys[®] Core and Visalys[®] Core are registered trademarks of Kettenbach GmbH & Co. KG.</p> <p>Note</p> <p>Not all Kettenbach products are available in all countries.</p> <p>For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to use by or on the order of a dentist.</p> <p>If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.</p> <p>Limitation of Liability</p> <p>Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from the use of this material, whether direct, indirect, special, in-tervening or consequential, even if the manufacturer has been advised, including warranty, contract, negligence, or strict liability.</p> <p>The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and instructions in connection with any use of the product.</p> <p>Updated: November 10, 2023</p>

ENGLISH
<p>Constituents</p> <p>The main components of the Visalys[®] Tooth Primer are water, acidic monomer (10-MDP) and HEMA.</p> <p>Trademarks</p> <p>Visalys[®] Tooth Primer, Visalys[®] Core Try In Paste, Visalys[®] Core and Visalys[®] Core are registered trademarks of Kettenbach GmbH & Co. KG.</p> <p>Note</p> <p>Not all Kettenbach products are available in all countries.</p> <p>For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to use by or on the order of a dentist.</p> <p>If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.</p> <p>Limitation of Liability</p> <p>Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from the use of this material, whether direct, indirect, special, in-tervening or consequential, even if the manufacturer has been advised, including warranty, contract, negligence, or strict liability.</p> <p>The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and instructions in connection with any use of the product.</p> <p>Updated: November</p>