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Indications for Use
Restoration primer for treatment of metal/silicate ceramics, oxide ceramic and composite surfaces

• to cement prosthetic restorations
• to repair fractured restorations with composite materials

Patient Target Group
Patients being treated in the context of a dental procedure.

Contraindications
Known allergies to methacrylic acid ester, 4-methoxyphenol, or other constituents of Visalys® Restorative Primer.

If there are no adequate means of ensuring drainage.

Warnings
Allergic reactions to the product cannot be ruled out in sensitive persons. Sensitization through contact with skin/mucous tissue is possible.

The use of hydrogen peroxide solutions, materials containing eugenol or retraction paste containing (iron/II) sulfate can impair adhesion.

Keep out of reach for children.

Caution of reactions
Do not use after the expiration date.

Do not swallow material. If the material is inadvertently swallowed: Consult a medical doctor if problems arise or persist.

Take the appropriate protective measures (e.g. wear gloves or safety glasses). Avoid contact of skin and mucous tissue with the material.

Avoid contact with the eyes. If there is accidental contact with the eyes: rise immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Avoid contact with clothing, as permanent damage to clothes cannot be excluded. Safety data sheets are available at www.kettenbach.com.

General Notes
Given refrigerator storage, the product has to have reached room temperature prior to use.

Visalys® Restorative Primer is intended for surface treatment of prosthetic restorations, indirect core build-ups and implant abutments and not for pretreatment of natural tooth hard substance.

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The adhesive surfaces must be clean and not contaminated, e.g. with saliva, blood or water, prior to applying Visalys® Restorative Primer.

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1.2. Adhesive surfaces of silicate ceramic (feldspar ceramic, glass ion and hybrid ceramics)

• Etch the surfaces with hydrofluoric acid according to the specifications in the instructions for use of the restoration material.

• Then rinse off thoroughly with water spray or clean in the ultrasonic bath.

• Dry with water /oil-free air.

1.3. Adhesive surfaces of composite

• Sand blasting of the surfaces with aluminum oxide powder (≤ 50 µm). Observe the restoration material manufacturer's blasting parameters.

• Then clean thoroughly with alcohol.

• Dry with water /oil-free air.

2. Drop Visalys® Restorative Primer into a preparation tray. Light protection is not necessary. Close primer bottles carefully after each use.

3. Apply Visalys® Restorative Primer with a disposable applicator or brush onto the adhesive surfaces and allow it to work for 60 sec.. Then blow out with dry air with a gentle jet of air. Any excess can be removed with a cotton pellet.

The adhesive bond to precious metal alloys can be enhanced by using a special precous metal primer.

Visalys® Restorative Primer should be processed quickly, as the constituents can evaporate.

Use as Primer to Repair Fractured Restorations

1. Roughen the entire fractured surfaces of the restoration material by blasting or using a suitable rotary instrument. Observe the restoration material manufacturer's blasting parameters and other measures, as applicable.

2. Clean out the fractured surface. Ensure a clean and dry operating field (cleaning with alcohol) during the entire application.

3. Drop Visalys® Restorative Primer into a preparation tray. Light protection is not necessary.

4. Apply Visalys® Restorative Primer with a disposable applicator or brush onto the adhesive surfaces and allow it to work in for 60 sec., and then blow out with dry air with a gentle jet of air. Any excess can be removed with a cotton pellet.

The adhesive bond to precious metal alloys can be enhanced by using a special precous metal primer.

Visalys® Restorative Primer should be processed quickly, as the constituents can evaporate.

5. Carry out the repair with a composite (e.g. Visalys® CemCore) according to the manufacturer's specifications.

Note
1. Part of the fractured surface consists of natural tooth substance, they have to be pretreated with adhesive. The specifications of the adhesive manufacturer must be observed.

Cleaning and Disinfection
The Visalys® Restorative Primer bottle can be cleaned with wipe disinfection using non-precious metal or oxide ceramics.

Disinfection of the bottle in accordance with national guidelines. Always standard practice hygiene measures.

Storage Conditions
Store dry, protected from sunlight. The product has to be kept in the refrigerator at 2°C/36°F–8°C/46°F before first use.

After the first use, Visalys® Restorative Primer has to be used within 6 months. Avoid extreme temperatures. Given refrigerator storage, the product has to have reached room temperature prior to use.

Do not use after the expiration date.

Constituents
The main components of the Visalys® Restorative Primer are ethanol, acidic adhesive monomers (10-MDP) and silane methacrylates.

Disposal
The empty bottles can be disposed in the household rubbish.

Trademarks
Visalys® Restorative Primer, Visalys® CemCore Try In Paste and Visalys® CemCore are registered trademarks of Kettenbach GmbH & Co. KG.

Note
Not all Kettenbach products are available in all countries.

For dental professionals use only. Caution: Federal (U.S.A.) Law restricts this device to sale to or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability
Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal basis asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are limited to the best of our knowledge, subject to technical changes within the framework of product development.

However, users must comply with and consider all recommendations and information in connection with any use of the product.

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