



**MORE VISALYS**

Dr. Tomaro also enjoys using Visalys Fill and Visalys Flow, two light-cured, nanohybrid composites that offer pleasant handling and easy shade matching in virtually any restoration. Scan the QR code to watch a quick video about what makes these materials unique.



**FOR MORE INFORMATION:**

877.532.2123  
kettenbach-dental.us

**KETTENBACH DENTAL**

# Visalys CemCore

*In just 5 short years, this 2-in-1 adhesive cementation and core buildup material has simplified and streamlined the restorative workflows of dental clinicians everywhere*

**T**hey say two heads are better than one and an extra set of hands makes light work. When it comes to dual benefits in dentistry, a product that can do two things with just one bottle is the ultimate symbol of simplicity and versatility. That's the definition of Visalys CemCore—a dual-curing adhesive cementation and core buildup material that offers an optimized bond, plus the stability needed to create a strong foundation.

Just ask Tony Tomaro, DDS, who has been using the material since it came to market. "One of the biggest issues with most resin cements is cleanup, especially when you're placing multi-unit restorations and it gets stuck in the gingiva," noted Dr. Tomaro. "In my office, we call Visalys CemCore the 'easy peel' because of how easy it is to clean up."

**One Material, Many Benefits**

Easy cleanup is merely the icing on the cake that is Visalys CemCore. Dr. Tomaro points to a long list of benefits gleaned from the material, including its versatility, strength, ease of use, and radiopacity.

"What's unique about Visalys CemCore is that it's multifunctional and can be used not only in



*"Visalys CemCore can be used with almost every restoration in dentistry!"*

—Tony Tomaro, DDS

cementation procedures, but in post and cores, which means you don't have to carry an inventory," he shared. "That's huge, and then there's the strength. Whether it's a retentive or non-retentive preparation, Visalys CemCore is a very strong cement!"

Moving right along to ease of use, Dr. Tomaro lauds the material's easy, step-by-step application. "The kit comes with just 3 components: Visalys Restorative Primer, Visalys Tooth Primer, and the cement itself. Both primers use the MDP monomer, which makes it easy to bond to zirconia."

**Use It Anywhere**

Launched in 2019 and deemed a *DPS* Best Product shortly after, Visalys CemCore was an instant hit among reviewing clinicians, several of whom decided to purchase or switch to the material after their trial period. Easy to place with a 5-minute self cure, it's designed to flow where it's needed but stay in place.

"The fact that it's dual-curing and self-etching makes it very unique," added Dr. Tomaro. "That's why you can use it post and core, because if you use a light-cure cement in a post, it doesn't travel the entire length all the way down. You need to have a dual cure, so it sets on its own."

He concluded, "Visalys CemCore can be used with almost every restoration in dentistry—hybrid ceramics, pure 100% ceramics, PFMs, metals, and also zirconium. So, it's very universal. That's a key aspect. And the fact that Kettenbach is a direct purchasing company makes it economically affordable."

**CHECKS EVERY BOX!**

- ✓ Multifunctional
- ✓ Easy handling and cleanup
- ✓ Extremely strong
- ✓ Dual-curing and self-etching
- ✓ Highly radiopaque
- ✓ Low film thickness